




DOCTORAIR

3D Massage Seat Premium  
**Instruction manual**  
(warranty included)

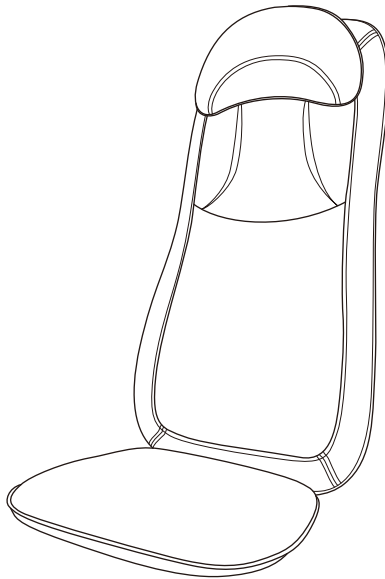

Thank you for purchasing this DOCTORAIR product.

- Please read this carefully before using this product.
- This instruction manual is also a warranty.
- Please keep this manual safe along with written information on the purchase date and the name of the purchase store.

The images in this manual are for illustration purposes only.

**Dream Factory Inc.**

## ■ Contents

**Safety precautions . . . . . 3~6**

• **Warning . . . . . 3~5**

• **Caution . . . . . 5~6**

**Performance and safety standard . . . . . 6**

**Purpose of use and effects . . . . . 6**

**Precautions before use . . . . . 6**

**Name of each part . . . . . 7**

**Remote control . . . . . 8**

**Preparation before use . . . . . 9**

**How to use . . . . . 10~13**

• **Start massage with an automatic course . . . . 10**

• **Start massage with a manual course . . . . 10~11**

• **Change the course in the middle of a massage . 11~13**

**Safety device . . . . . 10**

**Care and storage . . . . . 13**

• **How to care . . . . . 13**

• **How to store . . . . . 13**

**Troubleshooting . . . . . 14**



**Disposing of the device . . . . . 14**



**Warranty and after sales service . . . . . 15**




**Product specification . . . . . 16**

## ■ Safety precautions

Please be sure to read this carefully in order to prevent harm to the user or others, as well as damage to property. Please make sure you understand the following signs and symbols after reading this, and comply with these instructions.

<b>Warning · Caution</b>	
Explanation of the danger and damage caused by misuse.	
 <b>Warning</b>	<b>Incorrect use of product may cause fatal injury, or death.</b>
 <b>Caution</b>	<b>Incorrect use of product may cause injury, or property damage.</b>

<b>Example of signs and symbols</b>	
Explaining signs and symbols	
 Prohibited to disassemble	<p>⊘ indicates prohibition.            Words or pictures inside or near the ⊘ symbol shows prohibition.            The symbol on the left indicates that disassembly is prohibited.</p>
 Unplug the power plug	<p>● indicates that it is necessary to complete this action.            Words or pictures inside or near the ● symbol shows prohibition.            The symbol on the left indicates that the power cord should be unplugged.</p>

 <b>Warning</b>	
<div style="text-align: center;">             Prohibited         </div> <ul style="list-style-type: none"> <li>● This product is for household massage only. Please do not use it for other purpose.</li> <li>● Please do not use this product if massages are prohibited by your physician.            It may worsen your condition.            Examples: heart disease, acute illness, fever, abnormal blood pressure, thrombosis, severe aneurysm, acute varices, dermatitis, skin infection (including inflammation of the subcutaneous tissue), cervical spondylosis, alcohol or drug consumption, history of rashes due to chemical fibers, feeling ill, fatigue, etc.</li> <li>● Please do not use this product if you are unable to express your own intentions or operate operate the product.</li> <li>● Please do not perform other tasks while using this product. It may cause accident or injury.</li> </ul>	<div style="text-align: center;">             Prohibited         </div> <ul style="list-style-type: none"> <li>● Please do not use this product while wearing any jewellerys or scarf.            It may cause accident or injury.</li> <li>● Please do not use outlets and wiring instruments beyond the stated ratings or use the product outside of 100 V. It may cause fire.</li> <li>● Please do not let infants or children use, play, or stand on this product.            It may cause accident.</li> <li>● Please do not use this product on a heated carpet, near fire or while smoking a cigarette.            It may cause fire or mechanical failure.            Do not:           <ul style="list-style-type: none"> <li>● Use underneath bed coverings</li> <li>● Use under a blanket</li> </ul>           Failure to do so may result in fire, accidents or malfunction.</li> </ul>

## Warning



Prohibited

- Please do not try to remove the exterior covering from the body of the massager, and do not use the massager in a broken state. It is extremely dangerous.
- Please do not insert any sharp objects into the massager or charging port.  
It may cause fire, electric shock or accident.
- Please do not wrap the adapter cord around the massager or the body of the AC adapter.  
It may cause fire or electric shock.
- Please do not damage, modify, bundle up or bend the cord.  
It may cause fire or electric shock.
- Please do not pull hard on the cord when unplugging the AC adapter.  
It may cause a fire, electric shock or electrical short
- Please do not use any adapter other than the one provided, and do not use the provided adapter for any other product.  
It may cause fire, electric shock or mechanical failure.
- Please be cautious when using the massager on the neck. Do not use it on the front of the neck. It may cause accident or injury.



must obey





- People with the following conditions should consult a doctor before using this product:
  - implantable medical electronic devices that are susceptible to electromagnetic interference, such as pacemakers
  - malignant tumors
  - heart problems
  - severe blood circulation disorder in the legs
  - loss of temperature sensation
  - pregnancy or recent childbirth
  - perceptual disorder due to advanced peripheral circulatory disorder (such as diabetes)
  - wounds
  - weakness or needing rest
  - fever (over 38°C)
  - osteoporosis, spine fracture, sprained muscles
  - treated area
  - tenosynovitis, whiplash, cervical spine injury or herniated disc
  - abnormal spine or crooked bones
  - receiving medical treatment from physicians
  - problems in the body other than those listed above
- Check for any tears and damage to the cushion, buffer sheet, or fabric. If any problems are found, do not use the product. Contact at your purchased store or local supplier and ask for a repair. Continuous use may result in electrical shortage or injury.
- If any health problems occur while using this product, stop using it immediately and consult a doctor.
- If any problems are found, do not use the product; contact at your purchased store or local supplier and ask for repair.
- When inserting the AC adapter into an outlet or into the charging port of the main body, please insert it tightly into the base.
- Please confirm that all the function button and timer is in working condition before using the product.






must obey

- Please consult with a physician before using this product if:
  - You have undergone surgery, incorporated with metal, plastic parts or silicone implants
  - You have troubled skin such as atopic dermatitis, facial metalloids, inflamed areas, itching or burning areas, bruises, sensitive skin, keloids
  - You have pains on any part of your body.

## ! Warning

 Unplug the power plug	<ul style="list-style-type: none"> <li>● If the cord or plug of the AC adapter is damaged or the plug is loose in the outlet, stop using the product immediately. Unplug the AC adapter from the outlet and purchased store or local supplier for a repair.</li> <li>● If the main unit or AC adapter generates abnormal sounds, unusual smells or smoke, stop using it immediately and contact at your purchased store or local supplier. Continuous use may result in fire, electric shock or accident.</li> </ul>	 Touching with wet hands prohibited	<ul style="list-style-type: none"> <li>● The device is not waterproof; do not touch it with wet hands. It may cause electric shock.</li> </ul>
		 Prohibited near water	<ul style="list-style-type: none"> <li>● Please do not splash water or use in the places with high humidity such as bathroom, or outdoor. It may cause fire or electric shock.</li> </ul>
		 Prohibited to disassemble	<ul style="list-style-type: none"> <li>● Please do not disassemble or modify the device. It may cause fire, accident or mechanical failure.</li> </ul>

## ! Caution

 Prohibited	<ul style="list-style-type: none"> <li>● Please do not unplug the AC adapter while in use. It may cause accident or injury.</li> <li>● Please do not use on irritated skin.</li> <li>● Please use this device for no more than 20 min daily. Do not use it on the same part of the body for more than 3 min. It may decrease in effectiveness and worsen the symptoms.</li> <li>● Please do not use this device except for the purpose of massage. It may cause injury or sickness.</li> <li>● Please do not use this device with any other treatment device. It may cause accident or sickness.</li> <li>● Please do not use this device while putting weight on it. Do not use it under a blanket.</li> <li>● Please do not stand or sit on the main body. It may cause injury or mechanical failure.</li> <li>● Please do not use this on the floor. It may cause the unit to bend and suffer mechanical failure.</li> </ul>	 Prohibited	<ul style="list-style-type: none"> <li>● Please do not fall asleep while in use of this product. It may cause accident or injury.</li> <li>● Please do not drop the product, load it with heavy weight or allow it to be impacted. It may cause fire, electric shock or mechanical failure.</li> <li>● Please do not apply unreasonable force to the massager, such as by intentionally pushing strongly against it. It may cause injury, sickness or mechanical failure.</li> <li>● Please do not use the product while or after consuming alcohol. It may cause accident or injury.</li> <li>● Please do not use this product while eating.</li> <li>● Please do not use the product directly on the skin.</li> </ul>
		 must obey	<ul style="list-style-type: none"> <li>● When charging with tap, be sure to observe the rated capacity. Otherwise, it may cause fire or burnout.</li> <li>● Please do not use immediately after eating a meal.</li> </ul>



## Caution



Unplug the power plug

- Please unplug the AC adapter in case of a blackout.
- When not in use, unplug the AC adapter from the outlet. Moisture and dust can cause an insulation failure, fire or electric leakage.



Unplug the power plug

- When it's thundering, please unplug the power plug from the outlet immediately.
- When charging is complete, unplug the AC adapter from the outlet.

## ■ Performance and safety standards

### 1. Performance

Item	Specification	Remarks
Count of kneading: device with kneading function	Speed 1 33±3 times/min	Test method: JIS T 2002:2006 6.3
	Speed 2 40±3 times/min	
	Speed 3 47±3 times/min	
Count of tapping: device with tapping function	Speed 1 480±10% times/min	
	Speed 2 580±10% times/min	
	Speed 3 680±10% times/min	
Vibration frequency: device with vibration function	Low (1 motor ) 5500±10% times/min	
	Med (2 motor ) 5500±10%/ 5500±10% times/min	
	High (3 motor ) 5500±10%/ 5500±10% / 5500±10%times/min	
Moving range: device with moving treatment area	Speed 1 3.0±10% cm/sec	Test method: JIS T 2002:2006 6.4
	Speed 2 4.0±10% cm/sec	
	Speed 3 5.0±10% cm/sec	

### 2. Other performance

Item	Specification	Remarks
Timer	20min	Test method JIS T 2002:2006 6.5

## ■ Purpose of use and effects

Use it for general household as a substitute for massage.


Please do not use this for any other purpose than those listed above.

## ■ Precautions


- Please check for tears before using this product. If any problems are found, please do not use and contact at your purchased store or local supplier.
- Please confirm that all the function button and timer is in working condition before using the product.
- If any problems are found, do not use the product; contact at your purchased store or local supplier

# ■ Name of each part


## 【Front】

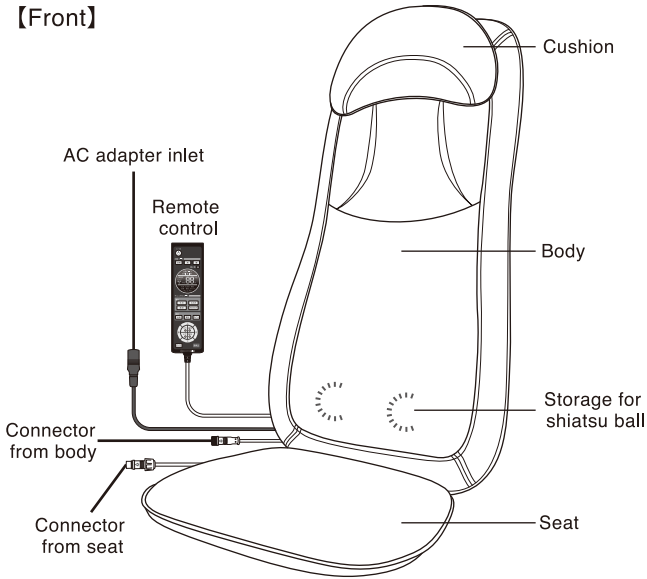




Power cord

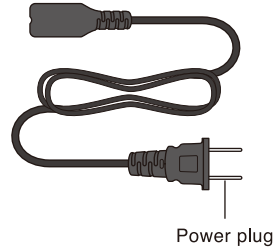


AC adapter

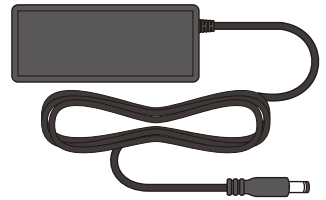

## 【Back】

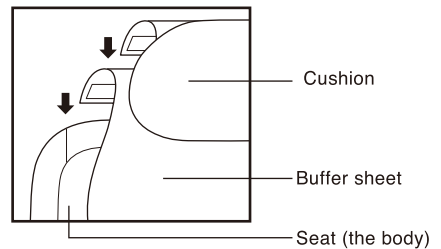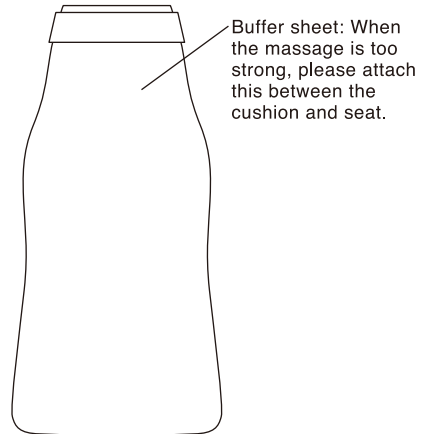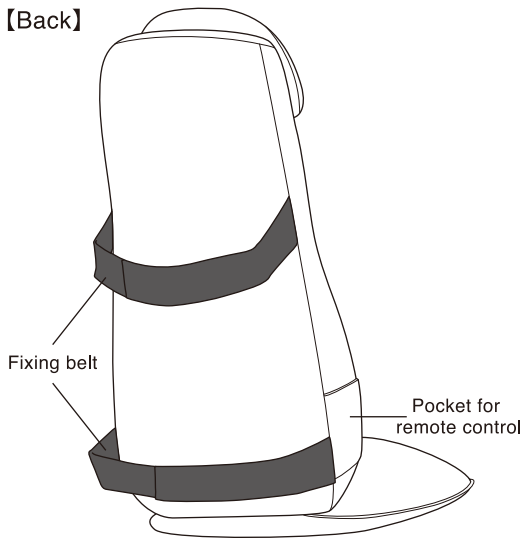

## Remote control

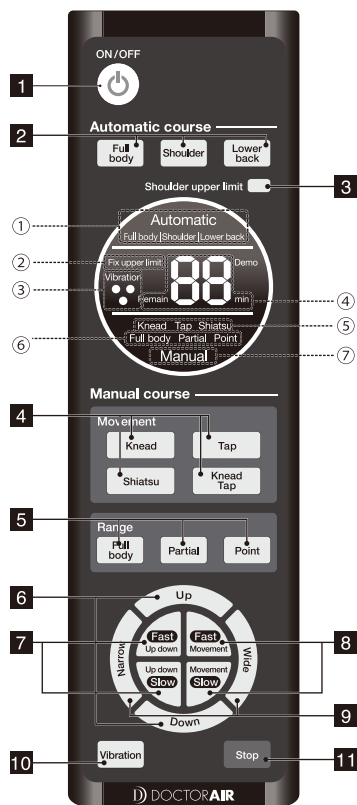

- 1** Power button  
Controls power on/off
- 2** Automatic course button  
Operates automatic massage course.
  - Full body : Massage full body
  - Shoulders : Massages full body, focusing on the shoulders
  - Lower back : Massages full body, focusing on the lower back
- 3** Shoulder upper limit button  
Set an upper shoulder limit, and the shiatsu ball will not go above the set position.
- 4** Manual course button
  - Knead : knead massage
  - Tap : Tap massage
  - Shiatsu : Shiatsu massage
  - Knead Tap : Knead and tap massage

- 5** Range button  
Set the vertical movement range of the shiatsu ball during manual course.
  - Full body : Vertical movement on full body.
  - Partial : Vertical movement within the range of 15 cm from the pressed point.
  - Point : Fix shiatsu ball at the pressed position.
 ※Unavailable to use point button during shiatsu massage
- 6** Up Down button  
Shiatsu ball go upward or downward.  
※Available during “partial” “point” of manual course is selected.
- 7** Up down fast slow button  
Speed up or down the vertical movement of shiatsu ball.  
※Unavailable to use during “point” of manual course is selected.
- 8** Movement fast slow button  
Speed up or down the kneading/tapping of shiatsu ball massage.  
※Unavailable to use during “shiatsu” of manual course is selected.
- 9** Wide narrow button  
Increase or decrease the width of shiatsu ball  
※Only available to use during “tap” and “shiatsu” of manual course is selected.
- 10** Vibration button  
Power on and off seat vibration and changing the strength of vibration.
- 11** Stop button

### 【Display】

- ①** Automatic course display  
Selected automatic course lights up.
- ②** Fix upper limit display  
Lights up when upper shoulder limit is selected.
- ③** Vibration display  
Display strength of vibration in three level.
- ④** Timer display  
Display remaining time of the massage.
- ⑤** Manual course movement display  
Selected movement of shiatsu ball during manual course lights up.
- ⑥** Range display  
Selected massage range during manual course lights up.
- ⑦** Manual display  
Lights up during manual course.




## ■ Preparations before use

### Attach seat

Body and seat are separated. Please attach seat to the body when use.


※The back part is where the cord and plug comes out

① Attach the seat body and the seat with a fastener. Ensure the zipper fully zipped.

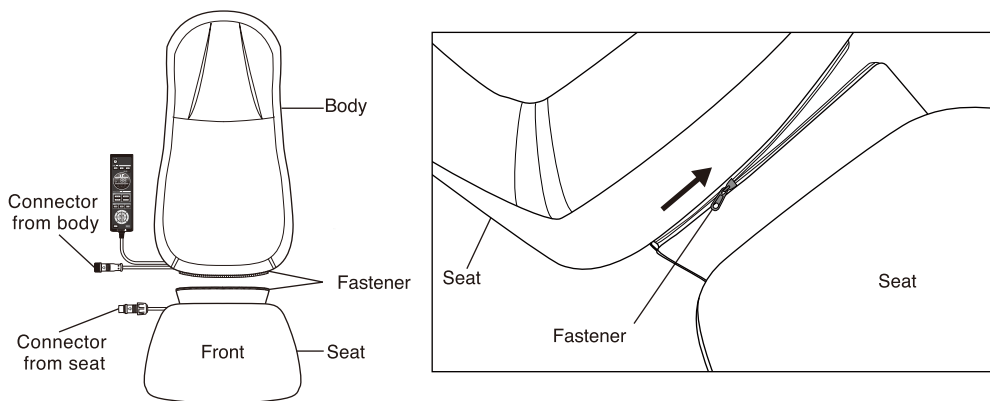


② Match the arrow mark on 'connector from seat' with the arrow mark on 'connector from body' and connect both together.

※ Firmly insert the connector from seat until the green silicone rubber is firmly against the connector from body.

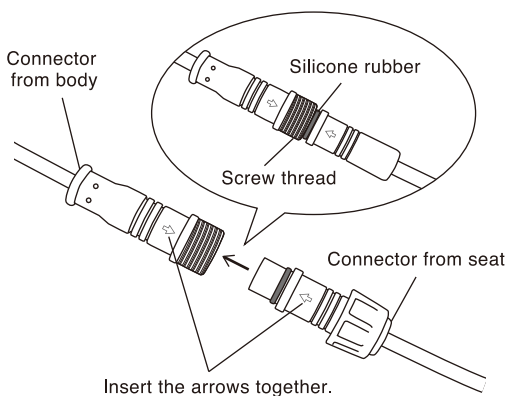


③ Turn the cap on the connector from seat side to firmly secure the connection.

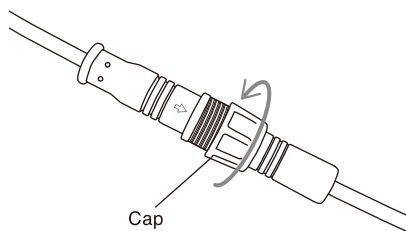


## To set up the massager:

- Place the massager's main body on a stable chair or sofa and attach the main body to a backrest, such as a chair with the fixing belt (Drawing 1).
- Connect the massager's main body, AC adapter, and power cord (Drawing 2).
- Disconnect the plug cap from the power plug and firmly insert it to the root of the household outlet. After plugging into the power outlet, make sure the power button on the remote control blinks blue.
- Attach the cushioning sheet if you find the massage to be too strong (Drawing 3).


Drawing 1

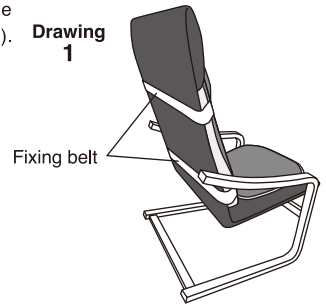

Drawing 2

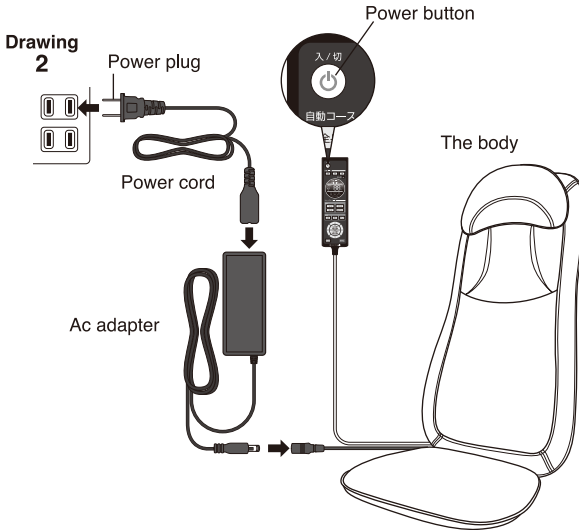

Drawing 3

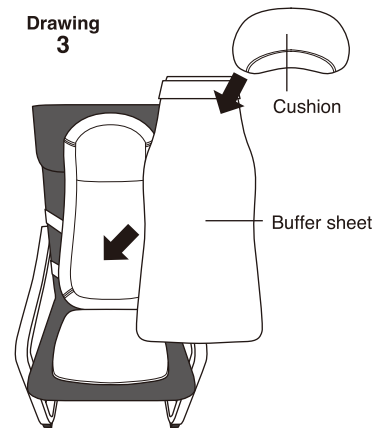


※Chairs for holding the main unit are sold separately.

### ⚠ Caution

- Do not use on chairs with casters or rotating chair.
- When using with a chair with a reclining function, be careful not to lean back during use.
- When using with a seat chair, do not use it on a hot carpet.
- Unit may scratch chair or sofa surfaces, place a rug beneath it to prevent this.
- When installing the chair or moving the massager's main body, do not drag the remote control or AC adapter. Store the remote control in the remote control pocket on the side of the main unit and remove the AC adapter from the main body of the massager.
- When inserting the AC adapter and power cord into an outlet or an adapter jack of the massager main body, insert it firmly into the base.

## How to use

### Start massage with automatic course

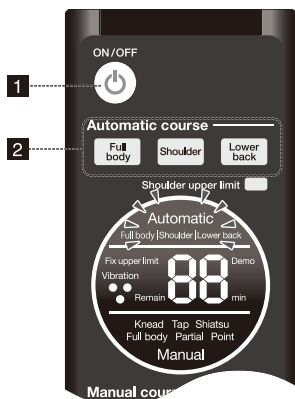

**1** The power button, timer, divider line, and border of the display section light up in blue, and the shiatsu ball moves to the top when the power button is pressed (standby mode).

**2** Choose from automatic courses “Full body,” “Shoulder,” or “Lower back” by pressing until the selected course lights up in blue.

**Full body** Full body massages from the neck to the lower back

**Shoulder** Shoulder massages from the neck to the lower back, automatically focusing on the shoulders.

**Lower back** Lower back massages from the neck to the lower back, automatically focusing on the lower back.

**3** Massage will end automatically after 20 minutes.

When the massage is complete, the shiatsu ball automatically returns to its original position, and the power button flashes blue.

### Start massage with manual course



**1** The power button, timer, divider line, and border of the display section light up in blue, and the shiatsu ball moves to the top when the power button is pressed (standby mode).

**2** Choose from “knead,” “tap,” “shiatsu,” or “knead tap” and press the button of your choice. Your selection lights in blue on the display.

※When “knead tap” is selected, both “knead” “tap” light up in blue.

**Knead** Both sides of the shiatsu ball rotate at the same time

**Tap** Shiatsu ball repeats a rhythmic tapping.

**Shiatsu** Shiatsu ball moves vertically without rotating.

**Knead tap** Combination of knead and tap functions

**3** Choose massage range, from “Full body,” “Partial,” or “Point” and press the button. The selected range lights in blue on the display.

**Full body** Full body massages from the neck to lower back.

**Partial** Partial massages a 15cm range from the position where the button is pressed.

**Point** Point massage is pinpointed with a fixed shiatsu ball where the button is pressed.

※ Unavailable for use during shiatsu operation

**4** Massage will end automatically after 20 minutes.

When the massage ends, the shiatsu ball returns automatically to its original location, and the power button flashes in blue.

## Change operation in the middle of use


### ● To set shoulder position:

#### Shoulder upper limit



Upper shoulder limit: Pressing the button in the middle of a massage will prevent the shiatsu ball from moving above that position. Press the button again to cancel the upper shoulder limit. Cancel the set shoulder position by pressing the upper limit button or save the position until the timer turns the power off. Fix the upper limit on the display: it will light blue while setting upper shoulder limit.

### ● Change the position of the shiatsu ball

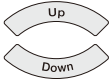

Pressing the button in the middle of operations moves the shiatsu ball vertically.



Shiatsu ball moves up by holding the button. Release the button at the location of your choice.



Shiatsu ball moves down by holding the button, Release the button at the location of your choice.

Up/Down button is available for use when in a manual course "partial" and "point".

### ● Change the width of the shiatsu ball path



Pressing the button in the middle of operation changes the width of the shiatsu ball path.



The width of shiatsu ball path narrows by pressing the button.  
The width of shiatsu ball path widens by pressing the button.

Choose from "wide" or "narrow" while selecting "tap" or "shiatsu" in a manual course.

### ● Change the speed of the shiatsu ball



The vertical movement speed of the shiatsu ball changes when you press the button in the middle of a massage.



Vertical movement speeds up by pressing the button.



Vertical movement slows down by pressing the button.

Unavailable for use when in a manual "point" course.  
Three levels of speed. The massage starts with level 2.

## ● Change the shiatsu ball's operation speed.



The shiatsu ball's operation speed changes as you press the button in the middle of a massage.



Operation speeds up by pressing the button.



Operation slows down by pressing the button

Unavailable for use when the manual "shiatsu" course is selected.


## ● Seat vibration on/off



Begin seat vibration by pressing the button in the middle of the massage.

Every time you press the button, the strength of vibration changes.

The display shows the vibration and strength (number of motors) via selected blue vibration lights while operating.

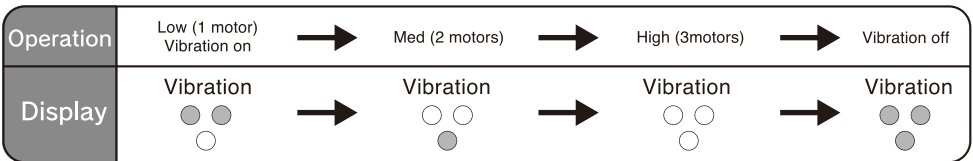

## ● Cancel massage in the middle



Massage ends and the shiatsu ball goes back to the storage position by pressing power button.

## ● If you feel Uncomfortable in the middle of use



All the operations stops if you press the stop button. To resume the massage, press any other button.



If you press the vibration button even during a sudden stop, the vibration of the seat will operate independently.

## ■ Safety device

A safety device is installed in this product. The safety device automatically turns off the power in the following cases:

- If the internal temperature of the motor or AC adapter exceeds the allowed temperature due to continuous use
- If strong pressure is applied to the kneading balls
- If any safety problems are detected

The safety device will allow the massager to work again after the inner temperature drops to its normal state. If the operation stops while in use and does not work when the power button is pressed, please let the device rest for 1 hour before turning on the power again

## Care and storage

### How to care

Body

- Please make sure to unplug the AC adapter from the outlet.
- Please clean with a soft, dry cloth.
- If severely dirty, please clean with a wet sponge or soft cloth follow by a dry cloth.
- Please do not use hot water or any detergent. They may cause mechanical failure, cracks and discoloration.

### How to store

- Please store only after unplugging the AC adapter from the outlet.
- Please clean off any dirt and dust and avoid storing in humid places such as bathrooms or in places with direct sunlight.
- Please do not store within reach of infants, children or animals.
- Please use a cover if the product will be stored for an extended period.

## Troubleshooting

Situations	Things to confirm
Power button does not work	<ul style="list-style-type: none"><li>● Check the AC adapter connection.</li><li>● The safety device may have been activated. Wait one hour and try again.</li><li>● Massage will not start in standby mode. Select the course and start the massage after exiting standby mode.</li></ul>
Suddenly stops working	<ul style="list-style-type: none"><li>● Massage stops automatically after 20 minutes, as set by a built-in timer.</li><li>● If the inner temperature becomes high or the shiatsu ball is under pressure, operations will be temporarily stopped by the safety device. Try again after one hour.</li></ul>
Power does not turn on after a massage	<ul style="list-style-type: none"><li>● The power button may not work while processing the end of a massage. Press the button after making sure all operations are ended</li></ul>
Generates noise	<ul style="list-style-type: none"><li>● This product generates noise due to its structure. This is not a mechanical failure.</li></ul>

Please contact at your purchased store or local supplier if further assistance is needed.

## Disposal

### ● Body

This product uses ABS resin and the fabric is safe. Please follow the regulations in your area to dispose of it.

### ● AC adapter

Please follow the regulations in your area to dispose of the adapter.

## ■ Warranty and after sales service

Please contact the store where the product was purchased or local supplier for information about use, care and repair.

### ● Written warranty

Please obtain a warranty from the store at the time of purchase and record the date of purchase and the store name. Keep the warranty in a safe place.

### ● Request for repair

Please see "Troubleshooting" (P.14) as a reference. If the situation does not improve or occurs frequently, remove the AC adapter and contact the store where the item was purchased or local supplier with the following information:

- Date of purchase
- Name of purchased store
- Current situation (details of problem)

- We will repair the unit in accordance with the provisions of the warranty card if a repair is requested within the warranty period. There may be charges even during the warranty period. Please read the warranty carefully.

**Warranty period: Written on the warranty**

- After the end of the warranty period, diagnose and repair charges apply, if the product is repairable. The repair price is consisting of the following contents.

**Technical fee: Diagnosis, repair, adjustment and inspection**

**Parts fee: Parts used for repair and auxiliary materials**


**Travel fee: Cost of dispatching an engineer to a repair location**

**Return shipping fee: Fee to return the repaired product**

※The minimum retention period of the parts necessary to maintain the function of the product is five years from the end of manufacture.

### ● Other questions

Please contact at your purchased store or local supplier for further assistance.

Please inspect 3D Massage Seat Premium from time to time		
 Inspection	<p>Do you see any of the following problems?</p> <ul style="list-style-type: none"><li>•Smoke, strange odor or noise</li><li>•AC adapter is extremely hot</li><li>•Does not charge when moving the cord</li><li>•Foreign material or water has gotten inside</li><li>•Deformation or damage on the body or cover</li><li>•Any other problem</li></ul>	<p>Please do not use</p> <p>Remove the AC adapter from the outlet and turn the power off. Please contact at your purchased store or local supplier to avoid a mechanical failure or accident.</p>

※For inquiries about mechanical failure and repair; Please contact at your purchased store or local supplier.

## Product specification

### Body

Name	
Model	
Category	
General name	
Types of medical device	
Certification number	
Purpose of use and effect	
Power supply	
Rated power consumption	
Count of kneading	
Count of tapping	
Vibration frequency	
Speed of shiatsu ball	
Width of shiatsu ball	
Timer	
Environmental temperature and humidity for operation	
Environmental temperature and humidity for storage	
Size	
Weight	
Accessories	
Material	
Manufacturing vendor	
Manufacturer	
County of manufacture	

### AC adapter

Model Name	YJS060A-1205000D
Rated input	AC100-240V~50-60Hz 1500mA
Rated output	DC12V 5000mA

※The performance and design of the product may change without notice.